Scan for more information

Check out our new learning platform Reliance U! [www.RelianceU.tech](#)

Reliance Orthodontic Products, Inc.
1540 W. Thorndale Avenue,
Itasca, IL 60143 USA
1-800-323-4348 | Made in USA | Rx Only
RelianceOrthodontics.com

EMERGO EUROPE
Prinsessgracht 20, 2514 AP The Hague,
The Netherlands

AUSTRALIAN SPONSOR
EMERGO AUSTRALIA
Level 20, Tower II, Darling Park, 201 Sussex St,
Sydney, NSW 2000, Australia

PORCELAIN CONDITIONER


INSTRUCTIONS FOR USE

CE
2797 **V1 NOV.2021**

RELiance®
ORTHODONTIC
PRODUCTS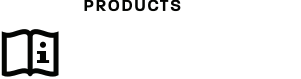

RelianceOrthodontics.com

ENGLISH TRANSLATION

INTENDED USE: Reliance Porcelain Conditioner is intended for use in the preparation of glazed dental porcelain for bonding. Porcelain Conditioner is a silane coupling agent which chemically unites with the silicon in the porcelain and the acrylic in chemical, light cure and dual cure adhesives.

INSTRUCTIONS FOR USE:

- Sandblast the porcelain surface. Rinse and dry.
- Apply one THIN coat of Reliance Porcelain Conditioner. Isolate and wait 2 full minutes. Thoroughly dry with air. If a warm air tooth dryer is utilized to dry, the wait time can be reduced to 1 minute.
- Apply 1 coat of Assure Plus, lightly dry with air and light cure for 10 seconds.
- Proceed with bracket placement.

⚠ **PRECAUTIONS:** May cause skin sensitization in susceptible persons. In case of contact with the skin, wash thoroughly with soap and water. Contact with other tissues: rinse area immediately with copious amounts of water for several minutes.

CAUTION: DO NOT REUSE IF BOTTLE BECOMES CONTAMINATED. A NEW APPLICATOR IS REQUIRED FOR EACH PATIENT TO AVOID CONTAMINATION RISK.

⚠ Porcelain Conditioner is flammable.

🌡️ **TEMPERATURE LIMITATION:** Store at room temperature (15-30°C)

WARRANTY: Reliance Orthodontic Products, Inc. recognizes its responsibility to replace products if proven to be defective. Reliance Orthodontic Products, Inc. does not accept liability for any damages or loss, direct or consequential, stemming from the use of or inability to use the products as described. Before using, it is the responsibility of the user to determine the suitability of the product for its intended use. The user assumes all risk and liability in connection therewith.

RX ONLY: Federal law restricts this device to sale by or on the order of a dental professional.

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