Scan for more information

Check out our new learning platform Reliance U! www.RelianceU.tech

Reliance Orthodontic Products, Inc.

1540 W. Thorndale Avenue, Itasca, IL 60143 USA

1-800-323-4348 | Made in USA | Rx Only RelianceOrthodontics.com

EMERGO EUROPE
Prinsessgracht 20, 2514 AP The Hague, The Netherlands

AUSTRALIAN SPONSOR EMERGO AUSTRALIA
Level 20, Tower II, Darling Park, 201 Sussex St, Sydney, NSW 2000, Australia

RELY-A-BOND® NO MIX ORTHODONTIC ADHESIVE


RELY-A-BOND® NO MIX ORTHODONTIC ADHESIVE

INSTRUCTIONS FOR USE

2797 **V1 NOV.2021**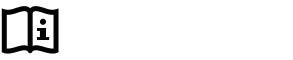

RelianceOrthodontics.com

ENGLISH TRANSLATION

INDICATIONS FOR USE: Rely•a•Bond® is intended for use in bonding metal, composite and ceramic orthodontic brackets.

CONTRAINDICATION: Rely A Bond is not indicated for bonding lingual retainers without bonding pads.

INSTRUCTIONS FOR USE:

PROPHYLAXIS: Prophy the enamel surface. Rinse and dry.

ETCH: Etch the enamel surface. Rinse and dry.

BONDING PROCEDURE: Apply a thin coat of primer to each etched and dried tooth in the area which the bracket is to be bonded, and the underside of each bracket base. With a spatula, apply a small quantity of adhesive paste to the underside of the bracket base and place on tooth. Within 25 seconds, position the bracket to desired angulation and press firmly. Excess adhesive can be easily removed from the periphery of the bracket base after all of the brackets are in place. An active archwire can be inserted 5 minutes after bracket placement.

⊠ **PRECAUTIONS:** Unpolymerized resins may cause skin sensation in susceptible persons. In case of contact with skin, wash thoroughly with soap and water. If accidentally instilled into eye, flush with copious amounts of water and seek medical attention immediately. Contact with other tissues: rinse spill area immediately with copious amounts of water for several minutes.

⊠ Rely A Bond Primers are corrosive.

⊠ **TEMPERATURE LIMITATION:** Store at room temperature (15-30°C).

TO REORDER:

REF # 1RPP 3.5g Rely•a•Bond® Paste in Push Syringe, REF # RPT-14g Rely•a•Bond® Paste in Tips, REF # RPJ-14g Rely•a•Bond® Paste in Jars, REF# 1RPPT – Rely A Bond Thin Paste in Push syringe, 3.5gm,

WARRANTY: Reliance Orthodontic Products, Inc. recognizes its responsibility to replace products if proven to be defective. Reliance Orthodontic Products, Inc. does not accept liability for any damages or loss, direct or consequential, stemming from the use of or inability to use the products as described. Before using, it is the responsibility of the user to determine the suitability of the product for its intended use. The user assumes all risk and liability in connections therewith.

RX ONLY: U.S. Federal law restricts this device to sale by or on the order of a dental professional.

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REF br. 1RPP 3,5 g Rely • a • Bond © pasta u potisnoj štrcaljki, REF br. RPT-14g Rely • a • Bond © pasta u vrhovima, REF br. RPJ-14g Rely • a • Bond © pasta u staklenki, REF br. 1RPPT - Rely A Bond tanka pasta u potisnoj štrcaljki, 3,5 gm,