



TKO™

COMPOSITE BITE TURBO GEL

INSTRUCTIONS FOR USE



RELIANCE®

ORTHODONTIC
PRODUCTS

[RELIANCEORTHODONTICS.COM](https://www.relianceorthodontics.com)

V1 JAN.2021





COMPOSITE BITE TURBO GEL

INDICATIONS FOR USE: TKO is a pink, flowable light cure composite gel for the creation of occlusal buildups (bite turbos) and as an acrylic appliance repair composite.

PERFORMANCE INDICATOR: Bonds to enamel for duration of treatment or a gel to repair acrylic appliances.

INSTRUCTIONS FOR USE:

INSTRUCTIONS FOR BITE TURBOS:

PROPHYLAXIS: Prophy the enamel surface. Rinse and dry thoroughly.



ETCH: Etch the enamel surface, rinse and dry.

APPLICATION OF GEL: Unscrew and remove the black storage cap from the tip of the TKO syringe. Snap the


needle tip dispenser in place and twist clockwise until needle is locked. Dispense the desired amount of gel onto the etched enamel surface and shape as desired. Light cure for 20 seconds from close range with a dental curing light of 1000MW2 or more.

INSTRUCTIONS FOR REPAIRING FRACTURED ACRYLIC:

With a brush or microbrush, apply a thin layer of gel to the clean and dry fractured site. Light cure for 20 seconds from close range with a dental curing light of 1000MW2 or more.

  **PRECAUTIONS:** Unpolymerized resins may cause skin sensation in susceptible persons. In case of contact with the skin, wash thoroughly with soap and water. If accidentally instilled into eye, flush with copious amounts of water and seek medical attention immediately. Contact with other tissues: rinse spill area immediately with copious amounts of water for several minutes.

 Corrosive

 Use of UV, halogen and LED curing lights can be associated with eye damage. The patient, doctor and the assistant should wear properly designed protective eyewear when using these light curing devices.



TEMPERATURE LIMITATION: Store at room temperature (20-25°C).

WARRANTY: Reliance Orthodontic Products, Inc. recognizes its responsibility to replace products if proven to be defective. Reliance Orthodontic Products, Inc. does not accept liability for any damages or loss, direct or consequential, stemming from the use of or inability to use the products as described. Before using, it is the responsibility of the user to determine the suitability of the product for its intended use. The user assumes all risk and liability in connections therewith.

RX ONLY: U.S. Federal law restricts this device to sale by or on the order of a dental professional.

NOTICE: Report any serious incident to Reliance and the competent authority of the state in which the user and/or patient is established.

INSTRUCTIONS FOR USE

V1 JAN.2021

CE
2797



RELIANCE®

**ORTHODONTIC
PRODUCTS**

🏭 Reliance Orthodontic Products, Inc.
1540 W. Thorndale Ave. Itasca, IL 60143 USA
1-800-323-4348 | Made in USA | Rx Only
RelianceOrthodontics.com

**AUSTRALIAN SPONSOR,
EMERGO AUSTRALIA,**

Level 20, Tower II,
Darling Park,
201 Sussex St, Sydney,
NSW 2000, Australia

EC REP

EMERGO EUROPE,
Prinsessgracht 20,
2514 AP The Hague,
The Netherlands

MD

Check out our new learning platform Reliance U!
www.RelianceU.tech

Scan for more
information

